


MATRIX SYSTEMS Strata-G™
Deep Margin Elevation24-25 & 27Reel Matrix™.26-27Slick Bands™ Tofflemire-style Matrices.28-29
Fusion™ Anterior.32-33VariStrip™ Anterior.31Blue View™ Cervical Matrices.37
RINGS Strata-G™ .6 3D Fusion™, 3DXR, 3D500, Gold, Original .7
BANDS Strata-G™. .9 Firm, Fusion, SXR. .10 SM, Gold, M, B .11
WEDGESStrata-G™
PRE-WEDGE SHIELD FenderWedge*
INTERPROXIMAL FINISHERS FitStrio™

POLISHING SYSTEMS Rally™ .20-2 eZr™ .22-2
MATRIX BAND FORCEPS Composi-Tight® Matrix
RING PLACEMENT FORCEPS Premier
CONTACT FORMERS Perform™
COMPOSITE INSTRUMENT Multi-Function Non-Stick
INFORMATION PAGES Strata-G™

Garrison® sectional matrix systems are the standard of care.

Tofflemire-style System

- Fails to restore proximal anatomy
- Thin contact at the marginal ridge
- Large food trap (below)
- Increased likelihood of fracture, occlusal interference, recurrent caries and periodontal disease

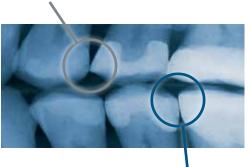




Strata-G[™] Sectional Matrix System

- + Operator-friendly retaining system
- + Naturally contoured bands
- + Anatomically correct contacts
- Contacts at the height of contour
- Rings produce optimal tooth separation for excellent, tight contacts

Now faster and more predictable!

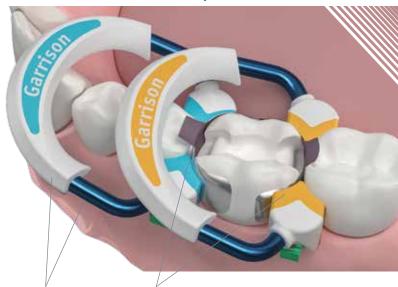

Reliable, tight contacts Drawn-wire

nickel titanium plus PEEK produces consistent separating pressure longer!

Easier, more predictable contours

Marginal ridge enhancements built into the matrix bands AND rings help provide that finishing flair!

Faster, less frustrating Class IIs

The Strata-G™ rings, bands and wedges combine for a 270° Seal™ for ultimate flash control. The rings won't spring off the tooth, and the amazing Wide Preparation ring makes tough wide preps no problem.

Garrison* sectional matrix systems are the standard of care.

Garrison® Sectional Matrix Cases

Great adaptation to the teeth eliminates the need for adjustment with polishers in the proximal areas.








Images courtesy of Dr. Javid Muraslov, Latvia

Wide preps can be handled in one simple step when you have the right (green) ring.



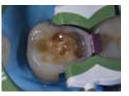



Images courtesy of Dr. Clarence Tam, HBSc, DDS, FIADFE, AAACD, Auckland, New Zealand

Back-to-back requires a shift in technique-we can show you how.*







Images courtesy of Dr. Anand Narvekar, Mumbai, India

Quadrants can be a challenge. Ease your concerns with tips and tricks from the pros.







Images courtesy of Prof. Nicola Scotti, DDS, PhD Torino, Italy

*To see full writeups, cases and reviews go to www.garrisondental.com/learning Do you have a case you want to share? Email it to us at contact@garrisondental.com.

The Award Winning Sectional Matrix System













TESTIMONIALS

David Naegele, DMD (The Strata-G[™] ring) created a much better seal on the prep for moisture control, and minimal, if any, flash. Strata-G[™] bands produced clear marginal ridges as I've never seen before. Finishing time for the restoration is GREATLY reduced.

of the (Strata-G[™]) rings help reduce flash, create a better seal around the box and my marginal ridge roll is excellent. Since

receiving Strata-G™

it is the only system

I use... I would cry

if someone took it

Nick Romenesko, DDS

The rubber feet

Steven Berwitz, DMD

I have used various sectional matrix systems from Garrison®, and I like this one the best. There is a significant reduction in flash and the contours of the restorations are ideal. The contacts made by this system are great!

away.



Strong tooth separation yet easier to open!

Nitinol drawn-wire nickel titanium is aligned at the molecular level for superior performance yet requires less effort to open. Garrison's proprietary StrataSet™ treatment maximizes the longevity of this advanced material.


More uses!

The PEEK ultra-durable reinforcement on the back of the ring allows Strata-G™ to retain a higher percentage of initial strength for much longer than non-reinforced rings. Proprietary StrataBond™ silicone/PEEK tips withstand hundreds of applications.

Wider indications—wider smiles! Conserve more of your patient's natural tooth se

patient's natural tooth structure with dentistry's premier Wide Preparation ring. The Strata-G™ Wide Preparation ring (green) provides a more conservative alternative for severely brokendown teeth.

Improved anatomy-Unbeatable grip

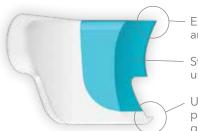


Enhanced marginal ridge guides and supports matrix bands.

 $StrataBond^{\mathsf{m}}\ silicone\ for \\ unsurpassed\ flash\ control.$

Ultra-Grip[™] retention extensions provide tenacious grip and are gentler to the papilla.

Strata-G[™] forms a 270° Seal[™] for ultimate flash control.

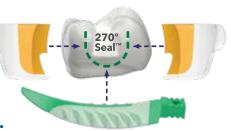



faster-more predictable-flash control



SGR-KSH-11 kit contains:

- 1 Universal/Short Ring (blue)
- 1 Tall Ring (orange)
- 1 Wide Prep Ring (green)
- 50 Assorted Matrix Bands
- 50 Firm Matrix Bands
- 10 VariStrips™
- 80 Assorted Ultra-Adaptive Wedges
- 12 Assorted Rally™ Polishers (pg 20-21)
- 1 Ring Placement Forceps (pg 37)
- 1 BFA-1 Band Placement Forceps (pg 36)
- 1 TN009 Multi-Function Instrument (pg 38)

Item

SGR-KSH-11

SGR-KSH-10 (BFA-1 and TN009 not in kit)

SGR-KSH-14 (TN009 not in kit)

For refill information on Strata-G™ rings, bands, and wedges, refer to pages 6, 9 and 12.

For more kit configurations and systems, go to garrisondental.com



Longer lasting by design.

Advanced materials and design combine to produce our longest lasting, most predictable separator ring ever!



SG500

All-New Separator Rings.

- The Strata-G[™] Universal/Short ring (blue) can easily grip the distal of the canine, short or malpositioned teeth and even primary teeth. The Universal ring can help you restore the widest range of cases in dentistry.
- The Strata-G[™] Tall ring (orange)
 provides ideal matrix band adaptation
 and separation on taller teeth.
 Additionally, it is perfect for stacking
 over the Universal ring (blue) for MODs
 or multiple tooth restorations.




Dentistry's Premier Wide Preparation Ring.

• The Strata-G™ Wide Preparation ring (green) allows you to conserve more of your patient's natural tooth structure. It provides a more conservative alternative than typical indirect restorations for severely broken-down teeth. Treat these cases as an extra-wide Class II rather than a complete crown prep with the confidence of knowing that you will achieve excellent contact and contour.



Composi-Tight. 3D Fusion. **

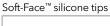



FX400 Short Ring 2/pk



FX500 Tall Ring 2/pk



FX600 Wide Prep Ring 2/pk

Composi-Tight. 3D XR **

The tenacious grip, along with enhanced Soft-Face™ technology and Dynamic Tip Angle™



3DXR Soft-Face™ Extra Retention Rings 2/pk

Composi-Tight.3D ******

The Soft-Face[™] 3D Ring provides maximum adaptation to the tooth



3D500 Soft-Face™ Rings 2/pk

Composi-TightGold. **

2x stronger than our Original G-Rings™



AU400 Standard Length Tines 3/pk



AU500 Long Length Tines 3/pk

Composi-Tight. **

The original G-Ring[™] that started it all—made of polished stainless-steel



G100 Standard Length Tines 3/pk



G200 Long Length Tines 3/pk



Case using Firm Matrix Bands

Dr. Anand Narvekar (Mumbai, India) utilized Garrison® Firm Bands for deep margin elevation in this challenging onlay case.



Figure 1.
FXH300 large molar band with extension was placed. The wedge was wrapped with multiple layers of teflon tape to sufficiently increase its size to allow it to seal the cervical portion of the matrix band in this extremely wide interproximal space.



Figure 2. Cervical area was built up with multiple layers of composite.



Figure 3.
The process was repeated for the adjacent molar. The stiff nature of Firm Bands allows apical pressure to be applied to the band without deformation of the matrix. It can be seated deeper than dead soft matrices for these types of procedures.



Figure 4. One-week post-bonding of onlays.



Anatomy top to bottom

Five sizes of super-thin stainless steel in both dead-soft and firm ensure you have the best matrix band to restore every case.



- Non-stick and color coded
- Easy placement tabs



- Dead-soft and only 0.0016" thin
- NEW pedodontic size!

Premolars **Pediatric** Premolars Molars with deep and small molars cervical preps SG Series Strata-G™ Strata-G[™] matrix bands with optimized marginal ridge thickness 0.0016" SG050 SG100 SG175 SG200 SG300 height 3.5 mm 4.5 mm 5.5 mm 6.5 mm 7.5 mm

Optimized marginal ridge

Strata-G[™] Bands provide shape and guidance for developing the marginal ridge of Class II restorations without blocking your access.

ProfileBooster™ subgingival apron

Achieving a natural emergence profile in challenging cervical areas is easy with Strata-G™ Bands-it's built right in!

Strata-G™ Matrix Bands

Assorted	Description Assorted Band Kit: 5 sizes Assorted Band Kit: 5 sizes			Qty. 400 200
	Standard (-M)		Bulk	
Item	Qty.		Qty.	
SG050	50		100	
SG100	50		100	
SG175	50		100	
SG200	50		100	

100





SG300

Band Comparison Chart

		Pediatric	Premolars	Premolars with deep cervical preps	Premolars and small molars	Molars	Molars with deep cervical preps			
ЭS	Composi-Tight* 3D Fusion™ Firm	Composi-Tight* 3	D Fusion™ Firm matrix	bands with eZ-Plac	ce™ technology for e	asier placement.			Standard (-M)	Bulk
erie					-	ed .		Item	Qty.	Qty.
, O	thickness							FXH100	50	100
S	0.0015"							FXH150	50	100
工		NA	FXH100	FXH150	FXH175	FXH200	FXH300	FXH175	50	100
\times	height	INA	4.3 mm	4.3 mm	5.3 mm	6.1 mm	6.1 mm	FXH200	50	100
ш	neight			w/ext 5.9 mm			w/ext 7.7 mm	FXH300	50	100
S	Composi-Tight* 3D Fusion™	C : T: L+* 2	DF : MF C	Daniela I	d				Standard (-M)	Bulk
Serie	3D Fusion ™	Composi-right 3	D Fusion™ Full Curve I	Bands have non-sti	ck coating and an oc	ciusai piacement tab		Item	Qty.	Qty.
			15					FX100	50	100
ζχ	thickness							FX150	30	60
	0.0016"					1		FX175	50	100
		NA	FX100	FX150	FX175	FX200	FX300	FX200	50	100
	height		4.0 mm	4.1 mm	5.0 mm	5.4 mm	6.0 mm	FX300	30	60
			w/ext 4.4 mm	w/ext 6.0 mm	w/ext 5.6 mm	w/ext 6.6 mm	w/ext 8.7 mm			
S	Slick Bands™ XR	Slick Bands™ X	(R Matrices have no	n-stick coating an	d Grab-Tabs™ for e	easier placement.			Standard (-M)	Bulk
<u>@</u> .		Silent Barras 7			u 3.45 .455 .6.	sacioi piacoiiioiia		Item	Qty.	Qty.
eri.								SXR100	50	100
Š	thickness 0.0016"							SXR150	30	60
8	0.0016							SXR175	50	100
<u> </u>		NA	SXR100	SXR150	SXR175	SXR200	SXR300	SXR200	50	100
S	height		4.6 mm	3.8 mm	5.4 mm	6.4 mm	6.4 mm	SXR300	30	60
				w/ext 5.4 mm			w/ext 8.6 mm			

Band Comparison Chart

								Darro	ГСОПРа	113011 01
		Pediatric	Premolars	Premolars with deep cervical preps	Premolars and small molars	Molars	Molars with deep cervical preps			
S	Slick Bands™	Slick Bands™ M	latrices have non-s	tick coating.					Standard (-M)	Bulk
<u>e</u> .								Item	Qty.	Qty.
Serie	.1.1							SM050	NA	60
Š	thickness 0.0016"	Carlot Control			9			SM100	50	100
	0.0010						139	SM150	30	60
\mathbb{S}		SM050	SM100	SM150	SM175	SM200	SM300	SM175	50	100
S	height	3.2 mm	4.6 mm	3.8 mm	5.5 mm	6.4 mm	6.4 mm	SM200	50	100
		w/ext 4.2 mm		w/ext 5.4 mm			w/ext 8.6 mm	SM300	30	60
S	Composi-Tight Gold*	The Gold band	ls are the same as t	he Composi-Tigh	t° bands.				Standard (-M)	Bulk
<u>.</u>	Gold	The Gold Balla	s are the same as t	ine composi rigii	t barras.			Item	Qty.	Qty.
er.							Company of the Compan	AU050	25	50
ζ	thickness 0.0015"		-					AU100	50	100
	0.0015						-	AU150	25	50
ارِــ		AU050	AU100	AU150	NA	AU200	AU300	AU200	50	100
\triangleleft	height	3.2 mm	4.6 mm	3.8 mm		6.4 mm	6.4 mm	AU300	25	50
-		w/ext 4.2 mm		w/ext 5.4 mm			w/ext 8.6 mm			
Ś	Composi Tight*	Composi-Tight	* bands are contou	ıred in 3 dimensic		_	w/ext 8.6 mm		Standard (-M)	Bulk
ies	Composi-Tight*	Composi-Tight	* bands are contou ct at the tooth's nat	ıred in 3 dimensic			w/ext 8.6 mm	Item	Standard (-M) Qty.	Bulk Oty.
eries		Composi-Tight		ıred in 3 dimensic			w/ext 8.6 mm	Item M050		
Series	thickness	Composi-Tight		ıred in 3 dimensic			w/ext 8.6 mm		Qty.	Qty.
\circ		Composi-Tight		ıred in 3 dimensic			w/ext 8.6 mm	M050	Qty. 25	Qty. 50
M Series	thickness	Composi-Tight		ıred in 3 dimensic		M200	w/ext 8.6 mm	M050 M100	Oty. 25 50	Qty. 50 100
\circ	thickness	Composi-Tight accurate contact	ct at the tooth's nat	ured in 3 dimension ural height of con M150 3.8 mm	ntour.	M200 6.4 mm	M300 6.4 mm	M050 M100 M150	Oty. 25 50 25	Qty. 50 100 50
\circ	thickness 0.0015"	Composi-Tight' accurate contact	ct at the tooth's nat	ured in 3 dimensic cural height of con M150	M175		M300	M050 M100 M150 M175	Oty. 25 50 25 50	Oty. 50 100 50 100
N N	thickness 0.0015" height	Composi-Tight accurate contact M050 3.2 mm w/ext 4.2 mm	M100 4.6 mm	med in 3 dimension wral height of con M150 3.8 mm w/ext 5.4 mm	M175		M300 6.4 mm	M050 M100 M150 M175 M200	25 50 25 50 25 50	50 100 50 100 100
N N	thickness 0.0015" height	Composi-Tight accurate contact M050 3.2 mm w/ext 4.2 mm	ct at the tooth's nat	med in 3 dimension wral height of con M150 3.8 mm w/ext 5.4 mm	M175		M300 6.4 mm	M050 M100 M150 M175 M200	25 50 25 50 25 50 25	Cty. 50 100 50 100 100 50
N N	thickness 0.0015" height Composi-Tight* Original	Composi-Tight accurate contact M050 3.2 mm w/ext 4.2 mm	M100 4.6 mm	med in 3 dimension wral height of con M150 3.8 mm w/ext 5.4 mm	M175		M300 6.4 mm	M050 M100 M150 M175 M200 M300	25 50 25 50 25 50 25 50 25 So 25	Cty. 50 100 50 100 100 50 100 50 Bulk
N N	thickness 0.0015" height Composi-Tight Original thickness	Composi-Tight accurate contact M050 3.2 mm w/ext 4.2 mm	M100 4.6 mm	med in 3 dimension wral height of con M150 3.8 mm w/ext 5.4 mm	M175		M300 6.4 mm	M050 M100 M150 M175 M200 M300	25 50 25 50 25 50 25 50 50 50 25 Standard (-M)	Cty. 50 100 50 100 100 50 8ulk Cty.
Series M S	thickness 0.0015" height Composi-Tight* Original	Composi-Tight accurate contact M050 3.2 mm w/ext 4.2 mm	M100 4.6 mm	med in 3 dimension wral height of con M150 3.8 mm w/ext 5.4 mm	M175		M300 6.4 mm	M050 M100 M150 M175 M200 M300	25 50 25 50 50 25 50 50 50 25 Standard (-M) Cty. 25	Cty. 50 100 50 100 100 50 8ulk Cty. 50
N N	thickness 0.0015" height Composi-Tight Original thickness	Composi-Tight accurate contact M050 3.2 mm w/ext 4.2 mm	M100 4.6 mm	med in 3 dimension wral height of con M150 3.8 mm w/ext 5.4 mm	M175		M300 6.4 mm	M050 M100 M150 M175 M200 M300	25 50 25 50 25 50 25 Standard (-M) Cty. 25 50	Cty. 50 100 50 100 100 50 Bulk Cty. 50 100
Series M S	thickness 0.0015" height Composi-Tight* Original thickness 0.0015"	Composi-Tight accurate contact M050 3.2 mm w/ext 4.2 mm The original ba B050 3.2 mm	M100 4.6 mm	M150 3.8 mm w/ext 5.4 mm shorter length.	M175 5.5 mm	6.4 mm	M300 6.4 mm w/ext 8.6 mm	M050 M100 M150 M175 M200 M300 Item B050 B100 B150	Cty. 25 50 25 50 25 50 25 Standard (-M) Cty. 25 50 25	Cty. 50 100 50 100 100 50 Bulk Cty. 50 100 50
Series M S	thickness 0.0015" height Composi-Tight Original thickness	Composi-Tight accurate contact M050 3.2 mm w/ext 4.2 mm The original ba	M100 4.6 mm nds have a slightly	M150 3.8 mm w/ext 5.4 mm	M175 5.5 mm	6.4 mm	M300 6.4 mm w/ext 8.6 mm B300 6.4 mm w/ext 8.6 mm	M050 M100 M150 M175 M200 M300 Item B050 B100 B150 B200 B300	Cty. 25 50 25 50 25 50 25 Standard (-M) Cty. 25 50 25 50	Cty. 50 100 50 100 100 50 Bulk Cty. 50 100 50 100 50

To order standard packs, add a -M after the item #.



Flexibility for a better cervical seal!

Strata-G[™] Wedges utilize Soft-Face[™] materials and advanced design to promote a better fit.



Stack them up.

The smooth tunnel design accommodates stacking of wedges. This feature is extremely helpful in cases of gingival recession or when simultaneous wedging from the buccal and lingual is desired.



Gentle on the papilla.

Smooth tunnel design and rounded edges glide over the papilla with less trauma.



Soft retentive fins smoothly fold down during wedge insertion and spring back when clear of the interproximal space, locking Strata-G™ Wedges in place.

A better fit by design.

The thoughtful design and advanced materials of Strata-G™ Wedges promote a better cervical seal.

The hollow underside sits lower and seals deeper interproximally, while the soft fins allow Strata- G^{TM} Wedges to adapt to concavities and irregularities.

Strata-G ™ Wedges

Description	Item	Qty.
Strata-G™ Wedge Kit	SGK4	400
Strata-G™ Wedge Kit	SGK4-M	200

		Standard		Bulk
Refills	Item	Qty.	Item	Qty.
X-Small Yellow	SGYL-M	50	SGYL	100
Small Blue	SGBL-M	50	SGBL	100
Medium Orange	SGOR-M	50	SGOR	100
Large Green	SGGR-M	50	SGGR	100

Helpful tip: If you have used Garrison® wedges in the past, you may find it helpful to use a size smaller than before due to the Strata- G^{TM} flared design.

Composi-Tight. 3D Fusion...

Ultra Adaptive Wedges

Combining Soft-Face[™] adaptive materials, a firm inner core and advanced mechanical features to produce a wedge that truly works. Every time.



A+Wedge 💝



GWAOR





GWAYL GWABL

Description	Item	Qty.
A+Wedge® Kit	GWAK4	200
		Bulk
Refills	Item	Qty.
X-Small Yellow	GWAYL	100
Small Blue	GWABL	100
Medium Orange	GWAOR	100
Large Green	GWAGR	100

Controls gingival bleeding with an aluminum sulfate coating that has an astringent effect on the gingiva, improves clarity of restoration area and eliminates need to apply an astringent separately

GWAGR

G-Wedg

Perfect wedge. Perfect contact.

If you prefer traditional wedge placement with either a hemostat or cotton pliers, G-Wedges give you the same great shape as WedgeWands*, just without the applicator wand.



WedgeWands. 🚼

WedgeWands® are designed to twist off the wand, but many prefer to leave it attached to help retract the cheek and for easy removal after the procedure is complete.



Great technique tips and product help are only a click away!

Join thousands of subscribers on our YouTube channel and get notified instantly when new content is posted.

The Garrison* YouTube channel is NOT a collection of infomercials. It's all about helpful information on a wide variety of topics. From short two-minute tips on how to reduce flash on Class IIs, to more in depth live patient case presentations, the Garrison* channel delivers great content.

Go to youtube.com/garrisondental or search Garrison Dental on YouTube and subscribe today!



Speaking of great content...

Get CE credits direct from Garrison®!

Go to the Learning Center at garrisondental.com to see a list of pre-recorded and upcoming live webinars—many with available CE.

Register for an online account on our website to sign-up for CE courses and manage your CE credits.



Garrison Dental Solutions
Nationally Approved PACE Program Provider for FAGD/MAGD credit.
Approval does not imply acceptance by
any regulatory authority or AGD endorsement.
9/11/2022 to 8/3/12/2026.
Provider ID# 304999

FENDER WEDGE. Interproximal Tooth Shields

Pre-separate and Protect

FenderWedge® prevents damage to the adjacent tooth and gingiva that can occur during Class II preparations.

FenderWedge* is a combination of a steel plate and a plastic wedge. Inserted into the interproximal space, it protects the tissue and separates the teeth, thus simplifying the application of a matrix.

FenderWedge®

Description	Item	Qty.
FenderWedge® Kit	TSK01	80

		Standard		Bulk
Refills	Item	Qty.	Item	Qty.
X-Small Purple	TSPR-M	30	TSPR	60
Small Orange	TSOR-M	30	TSOR	60
Medium Green	TSGR-M	30	TSGR	60
Large Yellow	TSYL-M	30	TSYL	60





TSGR





Small (orange) TSOR

X-small (purple) TSPR



Clinical Tip





Place FenderWedge® prior to preparation to pre-separate and protect the adjacent teeth.



The plastic wedge—not just the metal shield—should firmly engage the interproximal surfaces of the teeth. This will provide the optimum preseparation force and hold the FenderWedge* securely in place (Figures 1 and 2).



Figure 3 clearly shows marks from the bur along the shield. Much better to have those on the shield than on the adjacent tooth!

Once the preparation is complete, remove the FenderWedge® and place your matrix.



Dentistry and photos courtesy of Dr. Rob Ritter, Jupiter, Florida

Visit garrisondental.com/ learning to see Dr. Ritter's full case presentation.

FenderWedge is NOT a sectional matrix system nor does it replace the use of a matrix band.



IPR Management in Clear Aligners Therapy

Dr. Tommaso Weinstein, DDS, PhD

Interproximal space management is a very important issue in clear aligners therapy. This case presented crowding in upper and lower arches. It was treated with clear aligner therapy through a digital planification.

The interproximal reduction (IPR) was carefully planned. The multiple thicknesses of FitStrip™ allowed IPR to be performed with predictability and control. The patient reported excellent comfort during the procedure, better than what was reported with traditional strips.















Fig 4-7. Digital planification, before (4 and 5) and after (6 and 7). Numbers refer to IPR required.







Fig 9. Clear aligners in place. Fig 10 and 11. Direct composites on incisal edges.


Fig 12 and 13. Final result in upper and lower arch.



FitStrips are a complete line of interproximal trimmers and finishers from the company that invented tight interproximal contacts, **Garrison*** **Dental Solutions**.



One handy handle! FitStrip's simple, attachable handle makes a world of difference for both clinician ease-of-use and patient comfort.

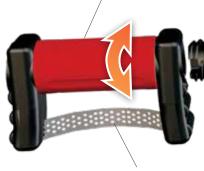

Ready for a marathon!

FitStrips are very durable and long lasting. They stay sharp and ready to go even after multiple autoclave cycles.

Starts bold - finishes clean

- 1. Medium grit for fast removal
- 2. Fine grit smooths things out
- 3. Super fine strips put the perfect finishing touch on things





Fit Strip.

Interproximal finishers with a Twist



- Primary Function: Contouring Composites and IPR up to .30mm.
- Durable diamond abrasive on just one side to protect adjacent teeth
- The FitStrip[™] serrated saw is the PERFECT instrument to separate accidentally bonded teeth and reduce over-tight contacts

FitStrip[™] Starter Kit

FPSK01 contains: 2 handles, 2 serrated, 4 single sided (1 of each .08 mm, .10 mm, .13 mm, .18 mm) 4 double sided (1 of each .11 mm, .15 mm, .21 mm, .30 mm)



Your Complete Solution for Interproximal Reduction (IPR)!

- Primary Function: IPR when .40mm and .50mm are needed, yet can still be used to contour composites
- IPR gauges for exact reduction
- Attachable handle for greater ease of use and patient comfort

Universal Kit

FPSK04 contains: 2 handles, 1 IPR gauge, 4 single sided (1 of each .08 mm, .10 mm, .13 mm, .18 mm) 6 double sided (1 of each .11 mm, .15 mm, .21 mm, .30 mm, .40 mm, .50 mm)



Subgingival FitStrips are the ideal solution for the cervical area!

- Primary Function: Use with Composites, both for contouring and accessing cervical areas to remove overhangs or excess cements.
- Only 2.5mm height keeps abrasive below the contact point
- Safety zone protects contacts during insertion
- Extra thin at .05mm

FitStrip™ Subgingival Finishing and Single Sided Kit

FPSK08 contains: 2 handles, 10 Strips: 2 subgingival super fine, 2 subgingival fine, 1 subgingival medium, 1 serrated saw, 1 each single sided .08 mm, .10 mm, .13 mm, .18 mm



FitStrips – a complete line of interproximal trimmers and finishers from the company that invented tight interproximal contacts, **Garrison Dental Solutions.**

Kits All kits come in a package of 10 FitStrips with 2 handles.

Description/Contents	ltem	Primary Use
Starter Kit for finishing/polishing 2 serrated 4 single sided (1 of each .08 mm, .10 mm, .13 mm, .18 mm) 4 double sided (1 of each .11 mm, .15 mm, .21 mm, .30 mm)	FPSK01 (shown on page 18)	Finishing composites
Universal Kit for IPR and finishing/polishing 1 IPR gauge 4 single sided (1 of each .08 mm, .10 mm, .13 mm, .18 mm) 6 double sided (1 of each .11 mm, .15 mm, .21 mm, .30 mm, .40 mm, .50 mm)	FPSK04 (shown on page 18)	Finishing composites and IPR
IPR Kit 1 IPR gauge 10 double sided (2 of each .11 mm, .21 mm, .30 mm, .40 mm, .50 mm)	FPSK05	IPR
Subgingival Finishing Kit 3 subgingival super fine, 3 subgingival fine, 3 subgingival medium, 1 serrated saw	FPSK07	Subgingival
Subgingival Finishing and Single Sided Kit 2 subgingival super fine, 2 subgingival fine, 1 subgingival medium, 1 serrated saw, 4 single sided (1 of each .08 mm, .10 mm, .13 mm, .18 mm)	FPSK08 (shown on page 18)	Subgingival and finishing composites

Refills		
	Single Sided	Double Sided
Super fine	FPSXFSS	FPSXFDS
15 Micron	0.08 mm thick	0.11 mm thick
	0.00 mm unek	o. i i iiiii anek
Fine	FPSFSS	FPSFDS
30 Micron	0.10 mm thick	0.15 mm thick
Medium		
	FPSMSS	FPSMDS
46 Micron	0.13 mm thick	0.21 mm thick
Coarse		
90 Micron	FPSCSS	FPSCDS
70 WILCTOTI	0.18 mm thick	0.30 mm thick
Serrated		
	FPS:	SAW
	0.05 m	m thick
Medium		FPSTMDS
46 Micron		
3		0.40 mm thick
Medium		FPSXTMDS
46 Micron		0.50 mm thick
		0.30 min tinck
Super fine	FPSGXFSS	
15 Micron	0.08 mm thick	N/A
Fine	Old Hill times	
	FPSGFSS	
30 Micron	0.10 mm thick	N/A
Medium		
46 Micron	FPSGMSS	N/A
40 MICION	0.13 mm thick	IN/A
FitStrip™ replacement han	dles (2-nack) EPS	L SHANDI F-2
Thomp replacement ham	(2-pack) 11 3	JI IAINULL-Z

IPR Gauge FPSGAUGE



Rally gets you to the finish line faster!

The composite finish line that is. One lightning fast step will put a beautiful satin finish on posterior restorations or add the gray high shine polisher for a lustrous gloss on facials and anteriors. And if you want to be quick off the starting line, use the blue coarse grit for rapid reduction and shaping. Rally Mini Polishers get you across the finish line and on to your next patient in record time!



Single patient Rally means that you get a clean, fresh, properly shaped polisher every time.



Mini and mighty!

Rally's small size and tapered shaft create a truly agile polisher capable of reaching the smallest anatomical features without blocking your visibility.



A clean get-a-way!

You don't want to leave a trail of crumbs or spattered polishing paste on the track. Rally's diamond impregnated high shine final polishers won't crumble during use and don't require paste to create a beautiful, durable shine.







Remove -Choose the Blue (silicon carbide) Polishers for fast removal and contouring



Smooth -Choose the Red (aluminum oxide) Polishers for smoothing and silky shine



Shine -Choose the Gray (diamond impregnated) Polishers for ultra high shine



Slow speed right angle latch shank. Use with water.

Rally™

Item	Description	Qty.
FPCK01	Composite Polisher Complete Kit	60 (10 each)

Item	Refills	Qty.
FPCC010	Blue coarse grit point composite polisher	30
FPCC040	Blue coarse grit cup composite polisher	30
FPCM010	Red medium grit point composite polisher	30
FPCM040	Red medium grit cup composite polisher	30
FPCF010	Gray fine grit point composite polisher	15
FPCF040	Gray fine grit cup composite polisher	15



Adjust and Polish Zirconia and Lithium Disilicate the **eZr** Way!

eZr. Intra-oral Adjustment:



- No chatter, chipping or microfracture Unlike diamonds bonded directly to a hard metal shaft, eZr's rubber substrate cushions the diamonds to prevent damage to the restoration.
- No heat build-up The carefully engineered substrate does not transfer heat during use.
- FAST! eZr's coarse grit blue grinders make your adjustments fast and convenient

eZr. Intra-oral Polishing:



- Unparalleled smooth polish - Don't take our word for it! -"The eZr system is the only one I've tried that polishes zirconia to smooth!" Todd Brower, DDS, Holland, Michigan.
- Increased physical stability - Highly polished zirconia and lithium disilicate is stable and nonabrasive to opposing dentition.
- No paste required Forget the mess and
 expense of polishing
 pastes. eZr[™] is all you
 need.
- No water required No need to use water
 when polishing.
 Water is only required
 for high speed.



Start by gross adjusting and contouring using the blue coarse grit grinders. Use the flat end taper for occlusal/labial/lingual/buccal surfaces and the disc for interproximal surfaces.



Use the green medium grit polishers for smoothing and pre-polishing. Use the point for the occlusal grooves, the cup for labial/lingual/buccal, and the disc for interproximal surfaces.



Use the orange fine grit polishers for high gloss polishing. Use the point for the occlusal grooves, the cup for labial/lingual/buccal, and the disc for interproximal surfaces.



Beautiful results!



Slow speed right angle latch shank Steam autoclaveable

Item	Description	Qty.
FPZK01	Complete system kit Gross removal, smoothing, pre-polishing, and high gloss polishing	2 grinders 6 polishers
FPZC030	Blue coarse grit disc Gross removal	5/pk
FPZC010	Blue coarse grit flat end taper Gross removal	5/pk
FPZM010 (not in kit)	Green medium grit small point polisher Smoothing and pre- polishing	5/pk
FPZM020	Green medium grit large point polisher Smoothing and pre- polishing	5/pk
FPZM030	Green medium grit disc polisher Smoothing and pre- polishing	5/pk
FPZM040	Green medium grit cup polisher Smoothing and pre- polishing	5/pk
FPZF010 (not in kit)	Orange fine grit small point polisher High gloss polishing	5/pk
FPZF020	Orange fine grit large point polisher High gloss polishing	5/pk
FPZF030	Orange fine grit disc polisher High gloss polishing	5/pk
FPZF040	Orange fine grit cup polisher High gloss polishing	5/pk



Direct cusp covering of vital element with a very deep cervical margin

Dr. Daniele Gensini

The case presents the need to seal the cervical margin, which is in a very deep position even if it can be isolated with a dam, and to recreate an adequate emergence profile. Thanks to the particular design of the Slick Bands™ matrices, this step was easy and made it possible to bring the restorative margin into a favorable position. Subsequently, after having recreated the direct covering of the missing cusp thanks to a previously created silicone index, the missing proximal wall was reconstructed with the aid of a Composi-Tight® 3D Fusion™ anatomical matrix, which was very useful for obtaining a correct emergence profile and an appropriate proximal contact point. Finally, the modeling of the restoration was completed and the correct occlusal contacts were checked.



Figure 1. Occlusal view of prepared tooth with cervical margin.



Figure 2. Slick Bands[™] matrix placement with correct emergence profile.



Figure 3. Highly filled flowable composite for reconstruction of the deep cervical margin.



Figure 4. Restorative margin in a favorable position and with correct profile.

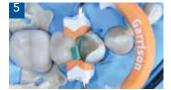


Figure 5. Correct adaptation of Composi-Tight® 3D Fusion™ matrix to the new restorative margin.



Figure 6. Lateral view of the correct emergence profile recreated.




Figure 7. Occlusal check.

Predictably relocate deep margins for better final restorations.

The conservative deep margin elevation procedure allows for the maximum conservation of natural tooth structure while helping to avoid more invasive procedures such as crown lengthening. Relocated supragingival margins facilitate easier scanning/impressions and proper isolation for improved bonding of inlays, onlays and direct final restorations.



Clinical Case: The margin elevation technique allows proper isolation and elevation of deep margin areas prior to restoring with an indirect partial coverage restoration (inlay or onlay). The use of the Garrison® Reel Matrix™ margin elevation band allows proper adaptation and seal of these deep areas so that a composite margin elevation can be performed.

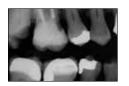

Pre-op X-ray shows the margin location for #17 subgingivally prior to elevation.




Seat the margin elevation band subgingivally. The narrowness of the matrix will allow it to slide subgingivally and seal the margin efficiently.



Seal the dentin and elevate the deep margin with restorative composite.



Remove the matrix and finish with a #12 blade to remove any flash. The deep margin has been relocated 1-2 mm supragingival for predictable isolation and delivery of an indirect bonded restoration.

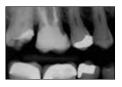

Post-op X-ray shows the completed margin elevation prior to final restoration being completed.

Photos courtesy of Matthew A. Nejad DDS, Beverly Hills, CA



Real Easy. Real Tight. Real Fast!

For all posterior restorations!

Real Easy:
The reels pop into the handle without any fuss! Press the reel into the handle and retract the grippers to lock the reel in place. Plus, if you use the regular matrices (yellow

hubs) they can be easily pushed through contacts.

Real Tight:

Tight contacts and excellent contour are what Reel Matrix is all about. These thin matrices are precontoured in three dimensions and when combined with anatomical wedges and separator rings, the result is amazingly tight contacts at the natural height of contour.

Real Fast:
Pre-loaded reels mean you can just lock them in and go. The retainerless design greatly speeds up quadrant work while improving visibility and patient comfort. Add a SG500 separator ring and you virtually eliminate buccal and lingual flash.



RM500H Margin elevation



RM200D 6.3mm deadsoft



RM100D

5mm deadsoft

RM100H 5mm regular



RM200H 6.3mm regular



RM100B 5mm blue



RM200B 6.3mm blue

Great for core build-ups







Margin elevation band is .002" thin. Super-thin dead-soft bands are .0012" thin. Regular bands are .0015" thin and can be placed through existing contacts. A Blue View cure-through option at .0029" thin is available if you prefer trans-enamel curing.

ReelMatrix

Real Easy. Real Tight. Real Fast!

Item Description	
------------------	--

DME-01 Deep Margin Elevation Kit

50 RM500H Margin Elevation band reels 1 RMH-100 Reel Matrix™ tensioning instrument

5 FXH150 3D Fusion™ Firm bands 10 FXH200 3D Fusion™ Firm bands 20 FXH300 3D Fusion™ Firm bands 20 FXYL 3D Fusion™ wedges 20 FXBL 3D Fusion™ wedges 20 FXOR 3D Fusion™ wedges 20 FXGR 3D Fusion™ wedges

RMK05

Contains 80 matrices

(20 RM500H, 10 ea. RM100D. RM200D, RM100H, RM200H, RM100B, RM200B) 1 tensioning instrument

1 PerForm[™] instruments (set of 2)

Refills		Qty.
RM500H	Margin elevation band	50
RM100D	5mm dead-soft matrices	50
RM200D	6.3mm dead-soft matrices	50
RM100H	5mm regular matrices	50
RM200H	6.3mm regular matrices	50
RM100B	5mm blue matrices	50
RM200B	6.3mm blue matrices	50
RMR100	5.9mm empty reel	100
RMR200	6.7mm empty reel	100
RMH-100	Tensioning instrument	

IMPORTANT ACCESSORIES

PerForm[™] Proximal Contact Instrument (included in RMK05 kit)

Page 35



Non-stick technology developed by Garrison®: Modern bonding agents can adhere to uncoated matrix bands–Slick Bands™ slide right out.

Exclusive Garrison shapes!




Proprietary Right Curve[™] design.

The increased curvature of the **Right Curve™ matrix band** produces outstanding gingival seal and reaches over to adjacent teeth easier for broader, deeper contacts.

This creates
a significantly increased
funnel-shaped cone when
wrapped around a tooth. It's the
right choice for many restorations.

Margin Elevation Band

The deep margin elevation technique is a useful non-surgical approach to treating deep localized margins.

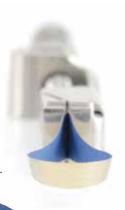

A margin elevation matrix will give you the best gingival adaptation and contour for the deep margin areas. It is trimmed in height to allow improved adaptation to deep margin areas.

Slick Bands[™] Tofflemire-style Matrices

Clinical Tip: Use dead-soft green bands when contact is COMPLETELY broken with adjacent teeth for maximum burnishability. When contact is NOT broken with adjacent teeth, use regular gray bands.

IMPROVED Dead-soft Bands **Regular Bands** with non-stick coating with non-stick coating #1 Universal for general restorations SMT200D10 SMT200H10 Right-Curve™ matrices provide a lower contact point and improved isolation. Excellent for SMT400D10 SMT400H10 all restorations. #13 Pedodontic for short and primary teeth SMT050D10 SMT050H10 #2 Subgingival for deep restorations SMT300D10 SMT300H10 Tofflemire-style matrices are .0014" thick See page 24 Margin elevation matrices are .002" thick for Dr. Gensini's SMT500H10 Margin Elevation Band Margin Elevation case.



Description	Item
Slick Bands Kit	SMTK06
300 Assorted Slick Bands	

100 G-Wedges PerForm instruments (1 set; shown on page 35) Standard (-M) Bulk

Item	Qty.	Qty.
SMT200D10	50	100
SMT400D10	50	100
SMT050D10	50	100
SMT300D10	50	100
	Standard (-M)	Bulk
Item	Qty.	Qty.
SMT200H10	50	100
SMT400H10	50	100
SMT050H10	50	100
SMT300H10	50	100
	,	
	Standard (-M)	Bulk
Item	Qty.	Qty.
SMT500H10	50	100
	SMT200D10 SMT400D10 SMT050D10 SMT300D10 Item SMT200H10 SMT400H10 SMT300H10 SMT300H10 Item	SMT200D10 50 SMT400D10 50 SMT050D10 50 SMT300D10 50 Standard (-M) Item Qty. SMT200H10 50 SMT400H10 50 SMT300H10 50 SMT300H10 50 SMT300H10 50 Standard (-M) Item Qty.

To order standard packs, add a -M after the item #.

Cases using anteriors/cervicals







Images courtesy of Dr. Zunzarren, DDS







Images courtesy of Dr. Abdi Sameni, DDS

VariStrip™

How can you produce a natural curvature for anterior composites and eliminate black triangles? Check out Garrison's curved "Varistrips" to recreate the natural interproximal anatomy for anterior teeth.

- Helps avoid incorrect flat embrasures, black triangles and food traps that can be caused by old-fashioned straight mylars.
- No need for wedges on subgingival fillings, as the contours naturally close off the apical aspects for you.
- Excellent for class IVs, diastema closures, etc.
- Variable height allows you to slide each strip in until it precisely matches the tooth height before cutting off excess.
- The blue tint does not affect cure, but ends the annoyance of losing clear strips on patient bibs or bracket tables.

Blue View VariStrips—a must-have super-simple product for every dental office.

Dr. Michael Curtis, Bridgeport, CT, USA





Clinical Tip

After using your preferred bonding procedure, place the restorative material into the preparation in advance of using the cervical matrix.

Place the cervical edge of the matrix against the tooth (Fig. 1).

While pressing the gingival edge of the matrix firmly against the tooth, roll the matrix onto the tooth (Fig. 2). Excess material will be expressed from the occlusal edge of the matrix rather than subgingivally, greatly simplifying clean up.

Cure directly through the matrix.

For product information on the cervical kit, refer to page 34.




David S. Hornbrook, DDS, FAACD International Lecturer and Private Practice, San Diego, CA

Curved Form/tapered: Provides the optimum curvature and band height for virtually every anterior restoration.

Blue Tint: Provides contrast between matrix and tooth structure without compromising composite polymerization.

Our Most Popular Anterior Matrix

BlueView.... VariStrip Contoured Anterior Matrices

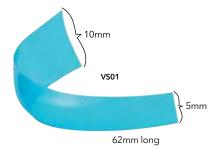

The Blue View[™] VariStrip[™] is a revolutionary contoured anterior matrix providing the optimum curvature and band height for virtually every anterior restoration. The 0.0020" thin plastic anatomical strip is tapered from one end to the other. You can place the strip interproximally and then slide it until the tooth height is perfectly matched. The pre-contouring easily recreates the occlusogingival anatomy, avoiding flat embrasures.

Blue View[™] VariStrip[™]

ltem	Qty
VS01	100
VS02	50



Case using the Fusion[™] Anterior Matrix System

Dr. JD Corey, DDS, Muskegon, Michigan



Figure 1. Flexible matrix systems have a tendency to deform subgingivally. This can lead to non-hygienic restorations and decay on adjacent teeth (as demonstrated in this pre-operative radiograph).



Figure 2. Rubber dam is in place and teeth have been pumiced before removal of the decay.



Figure 3. Decay has been removed, the preparations have been refined, treated with gluma, etched, and bonded. These steps were completed before placing the matrix bands and wedge to ensure proper coverage of the restorative materials to the preparations.



Figure 4. Stacked matrix bands have been placed to provide facial access and are ready to be engaged by the wedge.



Figure 5. Completed restoration after removal of the rubber dam.



Figure 6. A post-operative radiograph showing a broadened interproximal contact, improved emergence profile, and appropriate marginal integrity of the restorations.

To see more cases, go to the Learning Center at garrisondental.com/learning


Fusion Anterior Matrix System

The **Fusion™ Anterior Matrix System** from Garrison® gives you the materials and techniques to restore many types of anterior lesions with confidence.



ANK01 Contains:

- 50 short bands
- 50 tall bands
- 25 extra small wedges
- 25 small wedges
- 25 medium wedges
- 25 large wedges

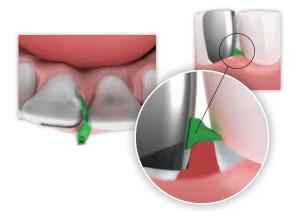





Fusion[™] Anterior Bands

		Standard		Bulk
Refills	Item	Qty.	Item	Qty.
Short	AN100-M	50	AN100	100
Tall	AN200-M	50	AN200	100



Learn more about the Garrison® Fusion™ Anterior Matrix System



Clinical Case using the Garrison® Fusion™ Anterior Matrix System with Dr. JD Corey



Fusion[™] Anterior Wedges

Description			ltem	Qty.
Fusion Anterior Wedge Kit			ANK4	400
Fusion Anterior W		ANK4-M	200	
	_	Standard	_	Bulk
Refills	Item	Qty.	Item	Qty.
X-Small Yellow	ANYL-M	50	ANYL	100
Small Blue	ANBL-M	50	ANBL	100
Medium Orange	ANOR-M	50	ANOR	100
Large Green	ANGR-M	50	ANGR	100

BlueView ::

Cervical Matrices

"I love the Blue View Cervical Matrix! It works so well. My assistants love watching me use this system. The patients love the excitement in our faces. It makes the process easy and fast. Gotta love that! We are also hooked on using the ReelMatrix. Thanks so much for making great products."

Cynthia Brattesani, DDS, San Francisco, CA, USA



Flexible

- adapts to individual anatomy for improved contours
- use it to directly compact the composite into the restoration
- thin edges allow excess composite to flow out for improved anatomy

Convenient!

- protects the restoration from contamination
- eliminates time spent hand sculpting
- eliminates air-inhibited layer
- reduces finishing time
- 6 sizes for a perfect fit

Blue Tint

- provides contrast between matrix and tooth structure
- does not compromise composite polymerization

KCMK2 INCLUDES

240 Assorted Matrices

Matrix placement Instrument

MATRICES INCLUDED:

Anterior/Premolar KCMA60 – 50 pcs

KCMA60 – 50 pcs Molar

KCMA70 - 50 pcs

KCMA10 - 35 pcs

KCMA20 - 35 pcs

KCMA30 - 35 pcs

KCMA40 - 35 pcs

Blue View[™] Cervical Matrices

ltem **Description** KCMK2 Assortment Kit w/instrument Refills Refill 100 pack KCMA10 Refill 100 pack KCMA20 KCMA30 Refill 100 pack Refill 100 pack KCMA40 Refill 100 pack KCMA60 Refill 100 pack KCMA70 KCMINST1 Placement Instrument

How **PerForm**[™] works:



STEP 1: Place the first composite increment. Press PerForm[™] into the uncured composite increment and either push (distal; fig. 1A) or pull (mesial; fig. 1B).



the curina on the Fresnel lens and cure





Proximal Contact Instruments

PerForm puts you in control.



- Improves contacts with any matrix system
- Control the height and breadth of contacts
- Increase the tightness of contacts
- Unique lens controls curing light diffusion
- Steam autoclave only

PerForm™

Item	Kit	Qty.
CFK01	PerForm (1/ea. size)	2/pk
CF100	1.1 mm Small Instruments	2/pk
CF200	2 mm Large Instruments	2/pk

"To ensure a tight contact a bulk fill flowable was placed in one proximal box at a time and the mylar held against the adjacent tooth with a PerForm™ proximal contact instrument. This ensures that the contact is tight and well-formed while light curing through it."

> Dr. Jack D. Griffin, Jr., DMD, MAGD Weldon Spring, MO

Composi-Tight. ** Matrix Forceps

Precise Placement, Unbeatable Grip.



Check out the GRIP! Effortlessly remove matrix

bands that otherwise would be a struggle.

BFA-1

Precise Placement: The tips have a 75-degree angle with an occlusal-gingival orientation, providing a more appropriate placement angle than cotton forceps. The petite, carbide tips allow the matrix to be grasped at its upper edge to prevent distortion of the interproximal area of the band.



Unbeatable Grip: The tungsten carbide tips provide exceptional grip and tactile control for tough access cases.

Matrix Forceps

BFA-1


Angled tip for occlusal placement

Garrison[®] Premier Ring Placement Forceps

- Extra durable
- Designed for all hand sizes and hand strengths
- Improved placement angle
- Traditional or butterfly ring placement

Ring Placement Forceps ***

Holds all your sectional matrix system rings.

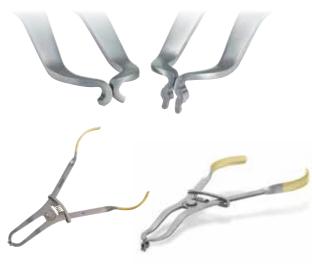

Garrison* Premier
Ring Forceps – Improved!
EXP01





FXP01





AUMRDF-100

Ring Placement Forceps

AUMRDF-100

Universal Forceps

for placing rubber dam clamps and rings UNVF-100



CompositeInstruments Nonstick titanium-nitride coated

Our multi-function instruments have double condenser configurations and carving blades on opposite ends. Now you can place, pack, shape and carve your way to perfectly contoured composite restorations without ever having to switch instruments!

- Large-diameter, lightweight handles for improved ergonomics.
- Provides non-stick surface for placement and shaping of composites.
- 40% smoother than stainless steel for a non-porous, non-stick surface.
- Withstands diverse cleaning/sterilization methods, including ultrasonic cleaning.



Item Description

TN001 Medium Blades

Placement of composite material Gingival retraction to sculpt under free margins Burnishing matrix bands



TN002 Medium Blade with Medium Plugger

Contouring small Class I, II, V Packing composite material into conservative preparations Burnishing matrix bands



TN003 Medium Blade with Large Plugger

Contouring large Class I, II, V
Packing composite material into large preparations

Packing composite material into large preparations

Convex plugger tip minimizes pullback of composite material



Item Description

TN004 Extra Thin Blades

Reach under margins to hone and sculpt Sculpting interproximally Packing cord in gingival crevice



TN005 Long Extra Thin Blades

Allows additional access for hard-to-reach posterior contouring and shaping Sculpting interproximally Packing cord in gingival crevice



TN006 Small/Medium

Multi-Function Plugger

Placement of composite material in smaller Class II and III Packing material toward gingival seats

Carving occlusal anatomy with acorn carver



****** Matrix Selection Guide

Being the Matrix Company, one of the most common questions we get is "which matrix do I use where?" While personal preference will drive much of the decision process, below is a chart that can provide a good starting point.

Procedure	Recommended Materials	Page	Special Notes
Distal of canine, short or malpositioned	Strata-G™ Sectional Matrix Kit	4–5	MOST POPULAR! Separator rings designed to hold firmly on tough cases.
	Reel Matrix™—Metal or Blue View	26–27	Retainerless Tofflemire-style matrices, contoured and easy to apply.
	Slick Bands™ Tofflemire-Style Matrices	28–29	Traditional Tofflemire-style with non-stick coating.
Conservative Class II	Strata-G [™] Sectional Matrix Kit	4–5	MOST POPULAR! Superior contacts with minimal flash.
	Reel Matrix [™] —Metal or Blue View	26–27	Retainerless Tofflemire-style matrices, contoured and easy to apply.
	Slick Bands™ Tofflemire-Style Matrices	28–29	Traditional Tofflemire-style with non-stick coating.
Slightly wide and/or deep	Strata-G [™] Sectional Matrix Kit	4–5	MOST POPULAR! Superior contacts with minimal flash.
	Reel Matrix™—Metal or Blue View	26–27	Retainerless Tofflemire-style matrices, contoured and easy to apply.
	Slick Bands™ Tofflemire-Style Matrices	28–29	Traditional Tofflemire-style with non-stick coating.

Procedure	Recommended Materials	Page	Special Notes
Large Class II with missing cusps	Strata-G [™] Sectional Matrix Kit—Green Wide Prep Ring	4–5	EXCLUSIVE! Dentistry's first Wide Prep ring greatly simplifies these more challenging restorations.
	Reel Matrix [™] —Metal or Blue View	26–27	Retainerless Tofflemire-style matrices, contoured and easy to apply.
	Slick Bands™ Tofflemire-Style Matrices	28–29	Traditional Tofflemire-style with non-stick coating.
Small to medium size Class III	Fusion™ Anterior Matrix System	32–33	Bands and wedges with ideal gingival-incisal curvature for many types of anterior lesions.
	VariStrip™ Contoured Anterior Matrices	31	Pre-contoured in three dimensions AND tapered to provide the ideal band height and contour.
Large Class III or Class IV	Fusion™ Anterior Matrix System	32–33	Bands and wedges with ideal gingival-incisal curvature for many types of anterior lesions.
	VariStrip™ Contoured Anterior Matrices	31	Pre-contoured in three dimensions AND tapered to provide the ideal band height and contour.
Class V	Blue View™ Cervical Matrix	34	Contours and directly compresses resin while eliminating air-inhibited layer.
Crown build-ups, post and core	Reel Matrix™	26–27	Best choice. Margin Elevation band available for very deep buildups. Retainerless convenience.
	Slick Bands™ Tofflemire-Style Non-Stick Matrices	28–29	Margin Elevation band available for very deep buildups if you prefer a traditional matrix retainer.

Order Form

Fax: 616.842.2430

Item Number	Description	Quantity	Price/Unit	Total Price
		Sales Tax		
		Total Price		

Doctor's Name			
Contact Name			
Address			
City	Province	Postal Code	
Phone	Fax		
Email			

All prices subject to change without notice. Appropriate shipping charges and sales tax will be added. Special offers not valid with any other advertised specials.

Customer Service Information

Call 888.437.0032 Toll Free

(USA & Canada)

616.842.2244

616.842.2430 Fax

Email gds@garrisondental.com

Online garrisondental.com

> Look for our Learning Center with clinical tips, case studies

and CE webinars.

Mail 150 DeWitt Lane

Spring Lake, MI 49456

USA

Distribution

Garrison is a global leader for practical and innovative dental products and accessories, with distributors worldwide.

We are pleased to announce that your favorite Garrison products are available through several full-service dealers in Canada.

To find a dealer, contact our headquarters in Spring Lake, Michigan. Phone: 888.437.0032 or Email: gds@garrisondental.com














Order Toll Free • Order by Fax • Order Online • Order by Mail

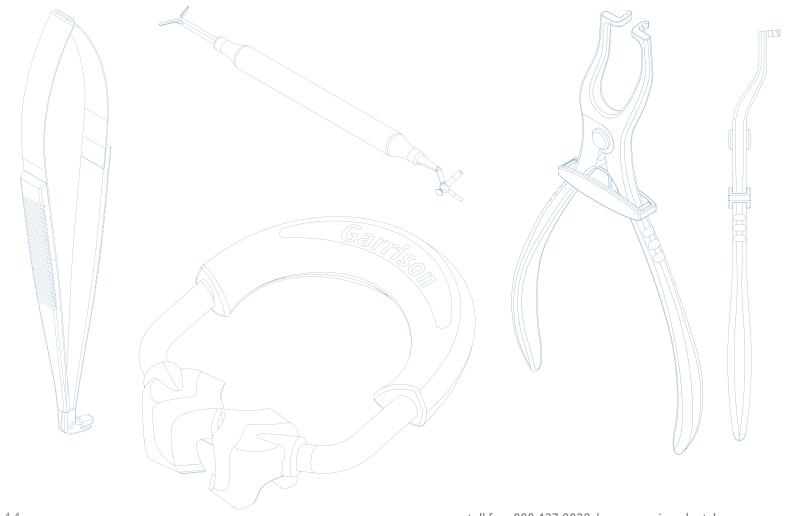

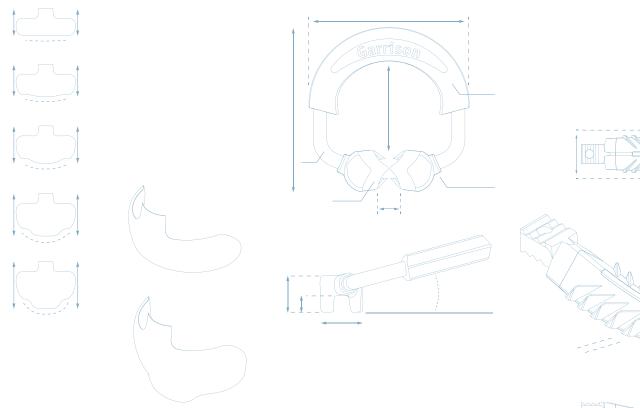

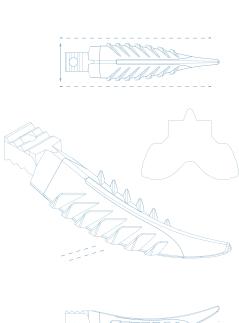



You can buy with complete confidence when you select Garrison products. We support all our products with expert customer service, outstanding warranties, and unrivaled satisfaction guarantees.

Please visit our website for details.



Distributed by Curion

EN: 1.800.667.8811

FR: 1.800.211.1200

curion.ca

Garrison

150 DeWitt Lane | Spring Lake, MI 49456 USA

toll-free 888.437.0032 | www.garrisondental.com

RLCACAT124