

## Adjust and Polish Zirconia and Lithium Disilicate the **Zr** Way!

## eZr. S Intra-oral Adjustment:



- No chatter, chipping or microfracture Unlike diamonds bonded directly to a hard metal shaft, eZr's rubber substrate cushions the diamonds to prevent damage to the restoration.
- No heat build-up The carefully engineered substrate does not transfer heat during use.
- **FAST!** eZr's coarse grit blue grinders make your adjustments fast and convenient.

## eZr S Intra-oral Polishing:



- **Unparalleled smooth polish -** Don't take our word for it! "The eZr system is the only one I've tried that polishes zirconia to smooth!" Todd Brower, DDS, Holland, Michigan.
- **Increased physical stability** Highly polished zirconia and lithium disilicate is stable and non-abrasive to opposing dentition.
- **No paste required -** Forget the mess and expense of polishing pastes. eZr is all you need.





## Making adjustments eZr S for you!

eZr<sup>™</sup> is a three-step diamond polishing system for the processing of zirconium oxide and lithium disilicate. The grinders and polishers are ideal for adjusting and correcting chairside restorations such as inlays, onlays, veneers and crowns.



Order #	Description	Use	Price
FPZK01	Complete system kit, 2 grinders and 6 polishers	Gross removal, Smoothing, Pre- Polishing, & High Gloss Polishing	\$271
FPZC030	Blue coarse grit disc gross removal, 5/pack	Gross removal	\$214
FPZC010	Blue coarse grit flat end taper gross removal, 5/ pack	Gross removal	\$162
FPZM010	Green medium grit small point polisher, 5/pack	Smoothing and Pre-Polishing	\$68
FPZM020	Green medium grit large point polisher, 5/pack	Smoothing and Pre-Polishing	\$102
FPZM030	Green medium grit disc polisher, 5/pack	Smoothing and Pre-Polishing	\$102



Start by gross adjusting and contouring using the blue coarse grit grinders. Use the flat end taper for occlusal/ labial/lingual/buccal surfaces and the disc for interproximal surfaces.



2 Use the green medium grit polishers for smoothing and pre-polishing. Use the point for the occlusal grooves, the cup for labial/lingual/ buccal, and the disc for interproximal surfaces.



3 Use the orange fine grit polishers for high gloss polishing. Use the point for the occlusal grooves, the cup for labial/lingual/ buccal, and the disc for interproximal surfaces.

Description	Use	Price
Green medium grit cup polisher, 5/pack	Smoothing and Pre-Polishing	\$102
Orange fine grit small point polisher, 5/pack	High Gloss Polishing	\$68
Orange fine grit large point polisher, 5/pack	High Gloss Polishing	\$102
Orange fine grit disc polisher, 5/pack	High Gloss Polishing	\$102
Orange fine grit cup polisher, 5/pack	High Gloss Polishing	\$102
	Green medium grit cup polisher, 5/pack Orange fine grit small point polisher, 5/pack Orange fine grit large point polisher, 5/pack Orange fine grit disc polisher, 5/pack	Green medium grit cup polisher, 5/packSmoothing and Pre-PolishingOrange fine grit small point polisher, 5/packHigh Gloss PolishingOrange fine grit large point polisher, 5/packHigh Gloss PolishingOrange fine grit large point polisher, 5/packHigh Gloss PolishingOrange fine grit disc polisher, 5/packHigh Gloss PolishingOrange fine grit disc polisher, 5/packHigh Gloss PolishingOrange fine grit cup polisher,High Gloss Polishing

Be sure to check out the website at **WWW.garrisondental.com** for info on other innovative Garrison products, demo videos and our 6-month money-back guarantee.

Distributed by Curion curion.ca | EN: 1.800.667.8811 | FR: 1.800.211.1200

