Digital Body Weight Scale
EB4074C

QUICK START GUIDE


EN

DE

ES

FR

IT

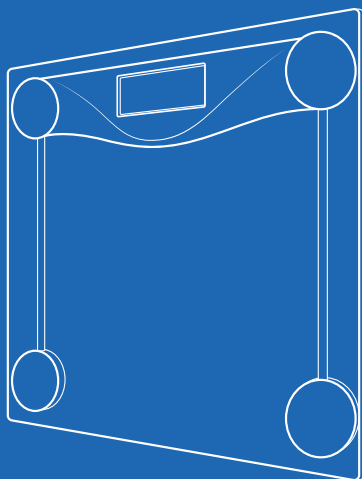QUICK START GUIDE

SPECIFICATIONS

Weight Capacity	180 kg / 28 st / 400 lb
Increments	0.1 kg / 0.2 lb
Weight Units	kg / st / lb
Platform	Tempered glass
Dimensions	30 x 30 x 2.4 cm / 11.8 x 11.8 x 0.95 in
Battery Type	2 x 1.5V AAA batteries
Auto-Off	15 seconds

PACKAGE CONTENTS

- 1 x Digital Body Weight Scale
- 1 x Quick Start Guide
- 2 x 1.5V AAA Batteries (Pre-Installed)
- 1 x Measuring Tape

TIPS

- To switch between kilograms (kg), stone (st), and pounds (lb), press the unit switching button on the underside of the scale.
- If your scale has been moved or flipped upside down, it needs to be calibrated.
 1. Step on the scale until digits appear on the display, then step off.
 2. The scale will show “{” while calibrating, then “0.0” when complete.

USING YOUR SCALE

Note: Before first use, remove plastic strip from battery compartment.

1. Place scale on a hard, flat surface (not a carpet or mat).
2. Step onto the scale. The scale will automatically turn on and start measuring your weight.
3. Stand on the scale until the digits on the display flash 3 times, showing your final measurement.

ONLINE MANUAL

Need more help? Scan here to view the manual for more information.

You can also type the following link in a web browser:
www.etekcity.com/support/manual/model/eb4074c

WARRANTY

Etekcity Corporation warrants all products to be of the highest quality in material, craftsmanship, and service for 2 years, effective from the date of purchase to the end of the warranty period. Warranty lengths may vary between product categories.

If you have any questions or concerns about your new product, please contact our helpful Customer Support Team.

CUSTOMER SUPPORT

Etekcity Corporation
1202 N. Miller St., Suite A
Anaheim, CA 92806
USA

Email: support.eu@etekcity.com

*Please have your invoice and order ID ready before contacting Customer Support.

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