

Topi-Pumps and Accessories



Assembly and Fill Instructions

Step 1. Fill bottle with the formula to be dispensed. (Filling methods shown are the syringe and Unguator® jar.) Do not over-fill the bottle.



Step 2. Place the actuator into the opening of the bottle and press firmly with the heel of your palm until the actuator clicks into place.



Step 3. Prime pump by pressing down on actuator until some of the contents are dispensed. The next pump will produce a full dose. When finished, detach and place link cap over tip of actuator. This prevents accidental discharge and makes actuator more child-resistant.



For hands-free application, apply dose to wrist and rub wrists together until formulation is fully absorbed into skin. Taking formulation orally is also made easy with the available oral adapter (see photo at top of page) which fits tightly over the actuator nozzle.

USP <795> creams, topical gels, ointments and pastes — When compounding semi-solid dosage forms, the compounder is to prepare an excess amount of total formulation to allow the prescribed quantity to be accurately dispensed. To ensure complete and proper dispensing, it is recommended that users over-fill the pump by at least an additional 10% of the total volume.

Note: Any application of prescription drugs should be evaluated by the pharmacist to ensure suitability. Keep prescription drugs out of reach of children.

* All trademarks remain property of their respective holders and are used only to directly describe the products being provided. Their use in no way indicates any relationship between Compounding Depot, Inc. and the holders of said trademarks. Visit <http://www.unguator.com> for more information regarding this product.

Topi-Pump[®]
Metered Dosing for Topical Pharmaceuticals **X**