

Fill bottle with the formula to be dispensed. (Filling methods shown are the syringe and Unguator®* jar.) After filling the Topi-Pump, tap the bottom of the pump on the counter to release trapped air. Removing the trapped air, is necessary to ensure accurate dosing. Do not over-fill the bottle. *Note: Any cream on the neck of the bottle will compromise dosing accuracy.



Place the actuator into the opening of the bottle and press firmly with the heel of your palm until the actuator clicks into place. *Note: With SleekLine pumps, there will be two distinct clicks.



Prime pump by pressing down on actuator until the first amount of the contents are dispensed. The next pump will produce a full dose. When finished, detach and place link cap over tip of actuator. This prevents accidental discharge, makes pumps more child-resistant, and keeps medication inside the nozzle from drying out.

For hands-free application, apply dose to wrist and rub wrists together until formulation is fully absorbed into skin.

*Taking formulation orally is also made easy with the available oral adapter (see photo at top of page) which fits tightly over the actuator nozzle.

[PDF Assembly Instructions](#)

USP creams, topical gels, ointments and pastes — When compounding semi-solid dosage forms, the compounder is to prepare an excess amount of total formulation to allow the prescribed quantity to be accurately dispensed. To ensure complete and proper dispensing, it is recommended that users over-fill the pump by at least an additional 10% of the total volume.

Note: Any application of prescription drugs should be evaluated by the pharmacist to ensure suitability. Keep prescription drugs out of reach of children.