

Warranty Details

Your Drive branded product is warranted to be free of defects in materials and workmanship for one year from date of purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the customer.

In the event of a defect covered by this warranty, we will at our option supply parts or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories and grips, which are subject to normal wear and need periodic replacement.

If you have any queries or require full warranty conditions, please contact your Drive stockist.

This does not affect your statutory rights.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the MHRA or competent authority of your Member State.

When the product has come to the end of its useful life, follow local recycling and disposal policies.



Com-Da Healthcare Co., Ltd.
1 Jinzhao Street, West District, Zhongshan City,
Guangdong Province, China



Osmunda Medical Technology Service GmbH.
Treskowallee 108, 10318 Berlin, Germany



Drive DeVilbiss Healthcare Ltd.
Sidhil Business Park, Holmfield,
Halifax, West Yorkshire,
HX2, 9TN, Great Britain

Drive DeVilbiss Sidhil Ltd.
4 Trench Road, Mallusk,
Newtownabbey, BT36 4TY,
Northern Ireland

info@drivedevilbiss.co.uk
www.drivedevilbiss.co.uk
Tel: +44 (0) 845 0600 333

drive

DeVilbiss®
HEALTHCARE



Quad Cane

Instruction Manual



Introduction



Please read these safety instructions carefully before use and keep them for future reference.

Intended Purpose:


This cane is a walking aid to assist those with reduced mobility.

Safety Precautions:

- Consult a healthcare professional before using any assisted mobility device.
- Not to be used by persons heavier than the maximum user mass.
- Not intended to support the full weight of the user.
- The cane is intended to be used on flat, level surfaces. Sloping and uneven surfaces will affect the stability of the cane and there is an increased risk of the user falling over.
- Not intended for use on ice or other slippery surfaces as it has the potential to slip.
- Do not use unless the sections are securely joined together by the e-clip.
- If the handle or tips become loose, or if the cane becomes damaged, stop using the cane immediately.



Maximum User
Mass Symbol

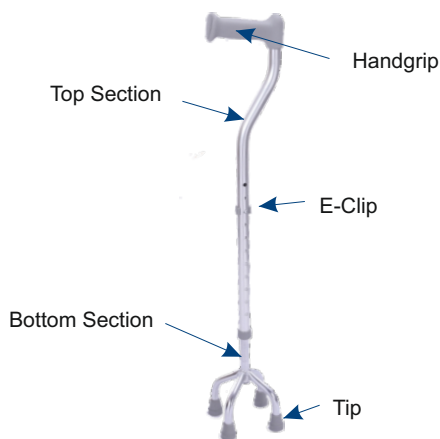

Height Adjustment

The ideal height of the quad cane is when the handgrip is level with the user's wrist. This should allow the user to have an upright posture with their elbow slightly bent.

To adjust the height of the cane, first pull out the E-Clip. Move the top section of the cane up or down until the required height is reached. Re-insert the E-Clip fully through the holes to secure the cane sections.

Specifications

Product Codes:	WS026	WS026L
Maximum User Mass:	127kg (20st)	
Cane Height:	69 - 95cm (27 - 37")	77 - 103cm (30 - 41")
Footprint:	Small	Large

Replacement Tip Code: FER16

Care & Maintenance

Check that the E-Clip is securely in position before every use.

Check that the handgrip and tips are secure.

Replace the tips if they become worn or loose.

Clean the cane occasionally with water and a mild detergent. Do not use abrasive detergents. Dry thoroughly before use.